

BENEFITS TO YOU



Lightweight



Silent



24/7 therapy



Easy to use



Full mobility



Comfortable



Pocket Size



Shower use



ADVICE FOR WASHING WITH NATROX®

The dressing used to cover the ODS needs to be waterproof to allow showering

Disconnect the ODS from the OG

The device should be placed in a safe area where it won't get splashed

Go for a shower, avoid direct exposure of the dressing to water



FREQUENTLY ASKED QUESTIONS



Why has my wound fluid increased?

Oxygen increases wound healing, as part of this process some people experience a rise in wound fluid. If you feel a build-up of fluid below your dressing or your dressing starts to leak it is important to let your clinical team know.

Will NATROX® cause any pain or discomfort?

NATROX® should not cause any pain or discomfort, some people report reduced wound pain when using NATROX®. If you experience any changes to your pain it is important to inform your clinical team.

Does NATROX® have any side effects?

There are no known side effects or risks currently associated with the use of NATROX®. However, if you experience any issues it is important to inform your clinical team.

How will I know if NATROX® stops working and what should I do?

If the green light on the NATROX® OG is flashing then oxygen is flowing, if the light stops flashing or turns solid red then:

- Check the battery hasn't become dislodged
- Swap the batteries over (a fully charged battery should last at least 24 hours)

If the light doesn't start to flash again contact your clinical team.

NATROX®
Oxygen Wound Therapy

For more information, visit us at:
www.natroxwoundcare.com

Inotec AMD @Natroxwoundcare Natroxwoundcare



PL_09/18 (V1)

Inotec AMD LTD Ground Floor Unit 7340, Building 7300 | Cambridge Research Park
Waterbeach | Cambridge | Cambridgeshire | CB25 9PD | United Kingdom



PATIENT INFORMATION LEAFLET

NATROX® Oxygen Wound Therapy



Designed simply to **HEAL**
WOUNDS

WHAT IS NATROX®?



NATROX® is a small portable device that generates oxygen which is delivered directly to the wound to help wound healing.



Your healthcare professional has chosen **NATROX® Oxygen Wound Therapy** because they think your wound might need extra oxygen to help the wound healing process.

WHY NATROX®?

Oxygen is critical in wound healing

Some wounds have a low level of oxygen

NATROX® provides continuous oxygen

HOW NATROX® WORKS?



NATROX® generates oxygen, and delivers it through a thin flexible tube to the **Oxygen Delivery System (ODS)** which is in direct contact with the wound surface.

OXYGEN GENERATOR (OG)

- Takes oxygen from the air
- Green flashing light indicates oxygen flowing




BATTERIES

- Battery slides into the OG
- When Charging:**
- Amber flashing light indicates the battery is charging
 - Solid amber light indicates the battery is fully charged



OXYGEN DELIVERY SYSTEM (ODS)




- Delivers oxygen to your wound
- Covered with a normal wound dressing

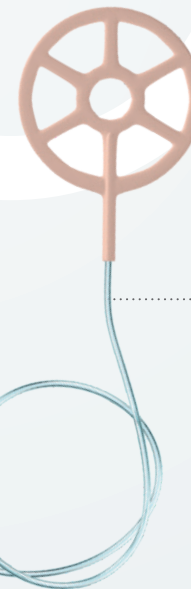

WHAT DO YOU NEED TO KNOW?

- A**lways have a battery on charge
- B**attery needs to be changed every 24 hours
- C**heck the green light is flashing on the OG

CHARGING THE BATTERY

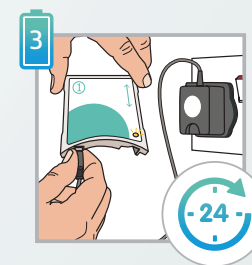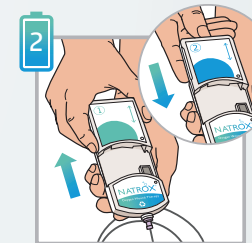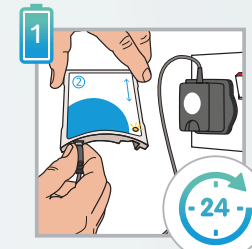

POSITION IN YOUR POCKET



WHAT TO EXPECT

